Clova Smart Rings USER MANUAL Bluetooth Connection Clova Wellness AI	1. Components and Package Contents Component Description Transparent Resin Oxygen/Heart Rate Sensor Charging port Test Status Indicator Packing List Smart Ring Charging cable Smart Ring USER MANUAL	2. Power On and Connect the App Connect the charging cable to power on the smart ring.Charging also begins automatically.	3. Connect the App 1. Install the app - Download the app or scan the QR code here. 2. Create an account - Open the app and register or sign in.follow the onscreen prompts. 3. Pair the ring - Enable Bluetooth, keep the ring near the phone and select Smart Ring in the Settings / bluetooth devices screen. Click on Search Clova Ring AI, select your device to connect. 4. Confirm sync - Wait for the app to show that the ring is connected and synchronizing. The data and chart start on home screen under "Your Ring"	4. Indicator Lights Red flashing - The ring is charging; Red off - Charging is complete; Red - oxygen-test is active; Green - heart-rate measurement is active. Charging Charged Over Blood oxygen Test Heart rate test	5. Wearing the Ring - Wear the ring on your preferred finger and choose a comfortable size. - Position the sensor area on the inside of the finger, toward he palm. - The ring should be snug enough to maintain contact without restricting circulation. Oxygen/Heart Rate Sensor	6. Care and Use Follow these precautions to protect the user and the device. Before use - Read and keep these instructions. - Keep the ring and charging accessories away from young children. - Use only a compatible charging cable and power source. - Do not use a damaged ring, cable, or charger. - Do not disassemble, puncture, crush, or modify the product. This product supports wellness tracking and is not a substitute for professional medical advice, diagnosis, or treatment.	7. Battery Safety Battery and handling - Avoid excessive heat, open flame, and prolonged direct sunlight. - Stop charging if the product becomes unusually hot, swollen, discolored, or emits an odor. - Do not place the product in household fire or expose it to extreme temperatures. - Clean with a soft, dry cloth. Do not use harsh chemicals. - Dispose of the ring and battery according to local electronic-waste rules.	8. FCC Compliance Statement This device complies with Part 15 of the FCC Rules. Operation is subject to two conditions - This device may not cause harmful interference; - This device must accept any interference received, including interference that may cause undesired operation. Changes or modifications: Any changes not expressly approved by the party responsible for compliance could void the user's authority to operate the device. Class B digital device notice: This device has been tested and found to comply with the limits for a Class B digital device under Part 15 of the FCC Rules. These limits provide reasonable protection against harmful interference in a residential installation. If this device causes harmful interference to radio or television reception, try one or more of the following - Reorient or relocate the receiving antenna; - Increase the separation between the equipment and receiver. - Connect the equipment to an outlet on a circuit different from the receiver. - Consult the dealer or an experienced radio or television technician. RF exposure: The device has been evaluated to meet general RF exposure requirements and may be used in portable exposure conditions without restriction.
--	---	---	---	--	---	--	---	---

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